



APAC Multipanel Cup for on Site urine screening

Tailor made to suit your requirements

The APAC Multipanel Cup can detect up to 20 different drugs; whether that are the common narcotics like Methamphetamines and Cannabis or others like New Psychoactive drugs.

Our idea is that you define the drugs you want to detect in the cup and we tailor make the configuration to suit your requirements; which could be the Australian standard six drugs or more. The cup can house up to 15 detection strips including adulterants, such as Creatinine, PH, SG, OX or GL.

Verified to AS/NZS4308:2008 and listed with TGA

The APAC Multipanel Cup has been tested with a Government NATA accredited laboratory and found compliant with the Australian Standard, appendix B. In addition the Australian Register of Therapeutic Goods have issued APAC Security an approval to supply the Cup as Medical Device, IVD Class 2, for non therapeutic usage.

Analysis on Site

Rather than sending all your samples to a toxicology laboratory for analysis, simply use the APAC Multipanel Cup for onsite screening and only send the non-negative samples for confirmation analysis. This will not only save you time but also significant budget savings. Instead of waiting several days for laboratory results, the APAC Multipanel cup will display a result after five minutes.

Easy to use

Collect the urine straight into the APAC Multipanel Cup, set the timer for five minutes and read the result(s).

Accurate

The APAC Multipanel Cup provides for a high level of correlation with GCMS / LCMS methods.

Long Shelf life

The APAC Multipanel cup offers two years shelf life from date of manufacturing. Storage temperature 2°C - 30°C.

Intended Use

The APAC Multi Panel Test Cup is a lateral flow chromatographic immunoassay for the qualitative detection of multiple drugs and drug metabolites in urine at the following cut-off concentrations:

Test	Calibrator	Cut-off (ng/ml)
Amphetamine (AMP)	D-Amphetamine	300
Barbiturates (BAR)	Secobarbital	300
Buprenorphine(BUP)	Buprenorphine	10
Benzodiazepines (BZO)	Oxazepam	200
Cocaine (COC)	Benzoyllecgonine	300
Methamphetamine (MET)	D-Methamphetamine	300
Morphine (MOP)	Morphine	300
EtG	Ethyl Glucuronide	300
Methadone (MTD)	Methadone	300
Methaqualone(MQL)	Methaqualone	300
Oxycodone(OXY)	Oxycodone	100
Tricyclic Antidepressants (TCA)	Nortriptyline	1,000
Marijuana (THC)	11-nor- Δ^9 -THC-9 COOH	50
Synthetic Cannabinoid (K2)	JWH-073/JWH-018	200
Tramadol (TRA)	Tramadol	200
Cotinine (COT)	Cotinine	200
Ketamine (KET)	Ketamine	500
Phencyclidine (PCP)	Phencyclidine	25
Propoxyphene (PPX)	Propoxyphene	300
Fentanyl (FYL)	Fentanyl	200

You Configure - We Deliver

Call for a free sample

t: +61 2 9986 2252

e: info@apacsecurity.com

w: apacsecurity.com

